

221 VIDEO and SCOPE SETUP FOR DR. CUADROS

[back to master list](#)

- Video tower marked clearly on Monitor: "Dr. Cuadros"
- Position Tower on left-hand side of room as you walk in facing the other wall so that Dr. Cuadros, anesthesia and scrub can see.
- Tower has printer and VHS recorder in line.
- Put new VHS tape in recorder, apply patient label to spine of tape cassette.
- ENT scopes and camera: 4mm, straight, 30°, 70°
- Omni-LapoTract
- Light cable 100%
- Separate mayo stand for scope (can also use for power equipment)
- De-fogger and lens cleaning pad

Rick Black

Mobile: 980-8480

Main: 1-800-421-0837 x6182

SONY PRINTER SETTINGS

multi pictures 2

print qty 1

input select RGB

If print freezes monitor → Menu -> PRN -> Function -> Autolive -> ON

menu | out | source | thru

MONITOR SETTINGS

RGB/component

"A"

HAND HELD CAMERA SETTINGS

White balance - use menu button

top (proximal) button - ACC 1 - controls printer capture

bottom (distal) button - ACC 2 - controls VHS record and stop

[back to master](#)